

INSTRUCTIONS FOR USE




NewGen Surgical Two-Compartment Tray

Please read all information carefully.

Intended Use

The NewGen Surgical Two-Compartment Tray is a bio-based, single-use, O.R. accessory designed to hold fluids and instruments during a surgical procedure.

www.newgensurgical.com/ifu

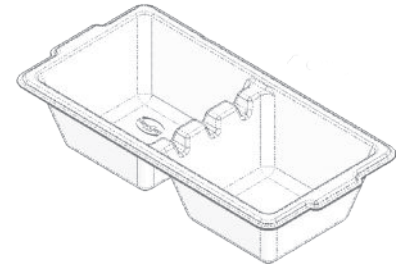

Warnings and Precautions

The NewGen Surgical Two-Compartment Trays are manufactured with bio-based materials. With these materials, some discoloration, texture variation, or fiber inclusions may be present. Cosmetic appearance does not affect the performance or sterility of the product.

To ensure performance, do not submerge in water or other liquids. Do not use excessive force.

This tray is manufactured for single-use only.

Do not reuse or reprocess. Reuse or reprocessing may compromise the structural integrity of the tray and/or lead to tray failure.

Store in a cool, dry place.

Do not use if product is damaged.

Do not put alcohol-based solutions into the tray.

Instructions for Use

Follow recommended healthcare procedures.

	Ethylene oxide sterilizable
	See Instructions for Use
	Do not reuse
	Batch code
	Use by YYYY-MM-DD or YYYY-MM
	Manufacturer

	Catalog number
	Do not use if package is opened or damaged
	Latex-free
	DEHP-free
	PVC-free

NGS2CT

NewGen Surgical, Inc.
San Rafael, CA 94901 USA
855.295.4500
www.newgensurgical.com



© 2023 NewGen Surgical, Inc.
Manufactured in China

45-0454-A