

# INSTRUCTIONS FOR USE




## NewGen Surgical Ring Basin 5500cc

**Please read all information carefully.**

### Intended Use

The NewGen Surgical Ring Basin is a bio-based, single-use O.R. accessory designed to hold fluids and instruments during a surgical procedure.

[www.newgensurgical.com/ifu](http://www.newgensurgical.com/ifu)

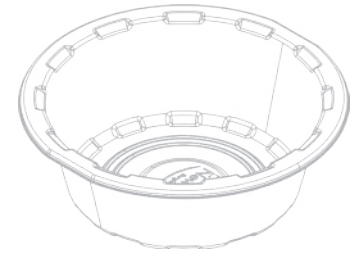

### Warnings and Precautions

The NewGen Surgical O.R. Basins are manufactured with bio-based materials. With these materials, some discoloration, texture variation, or fiber inclusions may be present. Cosmetic appearance does not affect the performance nor sterility of the product.

To ensure performance, do not submerge in water or other liquids. Do not use excessive force.

Do not use with Betadine Solution.

Do not put alcohol-based solutions into the basin.

This medical basin is manufactured for single-use only.

Do not reuse, reprocess, or resterilize. Reuse, reprocessing, or resterilization may compromise the structural integrity of the basin and/or lead to basin failure.

Store in a cool, dry place.

Do not use if product is damaged.

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

### Instructions for Use

Follow recommended healthcare procedures.

	Ethylene oxide sterilizable
	See Instructions for Use
	Do not reuse
	Batch code
	Use by YYYY-MM-DD or YYYY-MM
	Manufacturer

	Catalog number
	Do not use if package is opened or damaged
	Latex-free
	DEHP-free
	PVC-free
	European Conformity

**REF** NGS5500RBE

NewGen Surgical, Inc.  
San Rafael, CA 94901 USA  
855.295.4500  
[www.newgensurgical.com](http://www.newgensurgical.com)

**EC REP** Emergo  
Prinsessegracht 20 2514 AP,  
The Hague, The Netherlands

© 2023 NewGen Surgical, Inc.  
Manufactured in China  
US Patent D896369  
45-0401-A

